

# NeoFlow Bonnet

## Guidance for Use

Size baby for NeoFlow bonnet with sizing aid provided.






Place to nape of baby's neck and bring forward to brow.



Sizing down onto baby's brow to select the size and colour from the indicator.



Select the bonnet size as per sizing.

AMBT100/XXS	Pre-Term	up to 24cm	
AMBT100/XS	Extra Small	24-28cm	
AMBT100/S	Small	28-31cm	
AMBT100/M	Medium	31-34cm	
AMBT100/L	Large	34-38cm	
AMBT100/XL	Extra-Large	38-42cm	

Open bonnet and place to baby's nape.



Fit to cover ears and wrap snugly over baby's brow.



Secure with Velcro tabs.

