


# High Flow Oxygen Therapy – FD140

## Guidance for Use\*

1. Select correct size of AquaNASE® nasal cannulae.
2. Apply AquaNASE® nasal cannulae and secure neck strap as shown overleaf.
3. Insert humidification chamber onto heater plate and spike water bag.
4. Attach circuit as shown a-d.
5. Connect heater cable and temperature probes.
6. Turn on heater and select invasive mode.
7. Set flow at 20L/min and select prescribed FiO<sub>2</sub> for 2 minutes.
8. Increase flow to max 70L/min as required monitoring SpO<sub>2</sub> and respiratory rate.

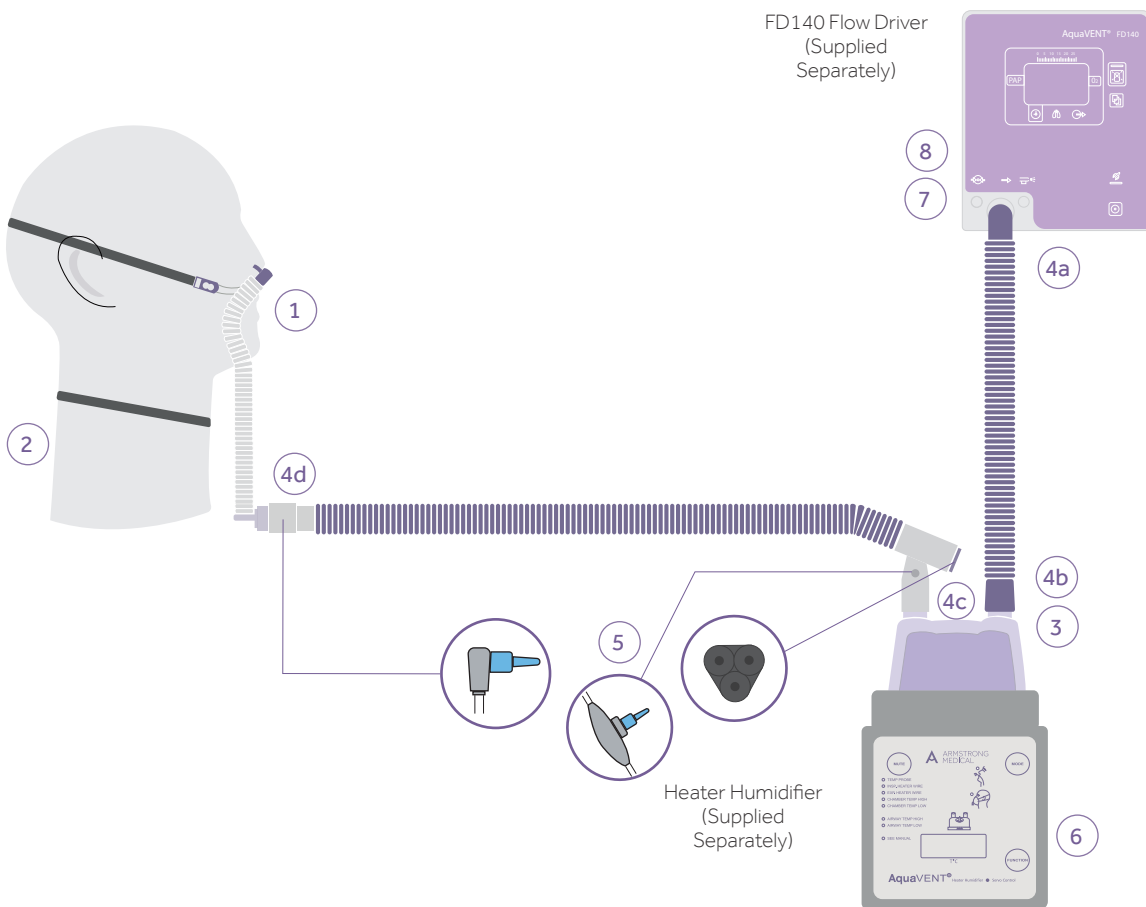

\*This guidance should be read in conjunction with the IFU supplied with the specific Breathing Circuit, Heater Humidifier, AquaNASE® nasal cannulae and the Flow Driver.

# AquaNASE<sup>®</sup>

## Fitting guide

### Step 1

Fit the looped neck support strap by placing over patients head and sliding down around the neck as shown.



### Step 2

Adjust the looped neck support strap by pulling one strap through the clamp as shown.



### Step 3

Place the elasticated strap around the patients head.



### Step 4

Adjust the ends of the elasticated strap to secure the device.



© Armstrong Medical Ltd. 2017



#### Armstrong Medical

Wattstown Business Park, Newbridge Road,  
Coleraine, BT52 1BS, Northern Ireland.

T +44 (0) 28 7035 6029 F +44 (0) 28 7035 6875

info@armstrongmedical.net

[armstrongmedical.net](http://armstrongmedical.net)

